




## **Urine/Catheter Bag Holders**

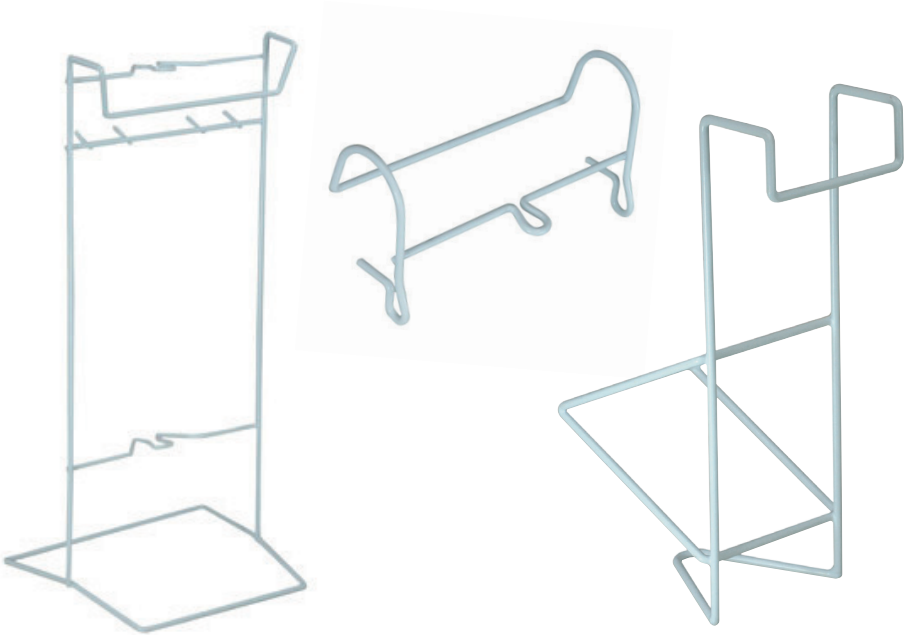

**VR270A: Urine/Catheter Bag Floor Holder**

**VR270B: Urine/Catheter Bag Hanging Holder**

**VR270C: Urine Hanging Holder**

## **Usage and Maintenance Instructions**

## INTRODUCTION

Thank you for deciding to purchase the Urine/Catheter Bag Holder from Aidapt.

## BEFORE USE

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product as it may compromise your safety, but contact your supplier for support.

## USING YOUR URINE/CATHETER BAG HOLDER

These Urine or Catheter Bag Holders will accept all popular bags and include hooks on which to hang the bag (from the hanging holes at either corner) they also feature integrated clip to hold the inlet tube.

VM270A is designed to rest on the floor, but can also be used as a hanging holder from the edge of the bed or bed rail.

VM270B is designed as a hanging holder from the edge of the bed or bed rail only.

VM270C is designed as a hanging holder to be slotted over the edge of the bed or bed rail.

## CLEANING

Our equipment can be cleaned and or disinfected as per MHRA guidance on decontamination MHRA DB 2003 (06)

Do not use abrasive cleaners as this could damage the surface of the equipment. For cleaning, wipe with a clean cloth soaked in detergent solution and wring out. Depending on the detergent manufacturer's instructions the solution may be heated.

Wipe all surfaces of the equipment and the cloth should be re-immersed periodically and the detergent changed frequently. The detergent should be rinsed or wiped off using a clean cloth immersed in clean water and wrung out.

To disinfect make sure the equipment has been cleaned and dried and then wipe over with a clean cloth and a compatible liquid disinfectant. After disinfecting clean with clean water and dry.

When using detergent or disinfectants then please refer to manufacturers' instructions and if unsure then consult your local Infection Control Specialist for advice.

When using moist heat the equipment can be held at 81° C for at least a minute to achieve disinfection.

## REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

**If in any doubt, please do not issue or use, but immediately contact your supplier for service support.**

## CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

## IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX**  
**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: [www.aidapt.co.uk](http://www.aidapt.co.uk)**  
**Email: [accounts@aidapt.co.uk](mailto:accounts@aidapt.co.uk) • [adaptations@aidapt.co.uk](mailto:adaptations@aidapt.co.uk) • [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**