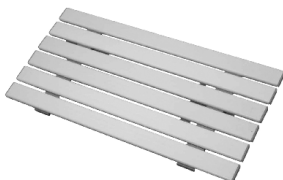
Medina Bath Seats and Boards

VR127/VR128/VR129
VR127A/VR128A/VR129A

VR124/VR125/VR126
VR124A/VR125A/VR126A

VR116/VR117/VR118
VR116A/VR117A/VR118A

Fixing and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Medina Bath Seat/Board from Aidapt. The Medina Bath Seat/Board range is manufactured from the finest materials and components available, when used correctly it is designed to give many years of reliable trouble-free service.

Please Note: Always consult with the prescriber/Physiotherapist if you are in any doubt that this product is suitable for your use.

NB. This equipment must be installed by a competent person, and a risk assessment may be needed as to the suitability of the equipment for the particular user

VR127/VR128/VR129	Medina Bath Seat	30 st (190 kg) weight limit
VR127A/VR128A/VR129A	Medina Bath Seat	30 st (190 kg) weight limit
VR116/VR117/VR118	Medina Bath Board	25 st (159 kg) weight limit
VR116A/VR117A/VR118A	Medina Bath Board	25 st (159 kg) weight limit
VR124/VR125/VR126	Medina Shower Board	25 st (159 kg) weight limit
VR124A/VR125A/VR126A	Medina Shower Board	25 st (159 kg) weight limit

Do not exceed the weight limit stated – doing so could put the user at risk.

BEFORE USE

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

FITTING BATH AND SHOWER BOARDS

1. Turn the board over with the cleats facing upwards (see fig. 1)

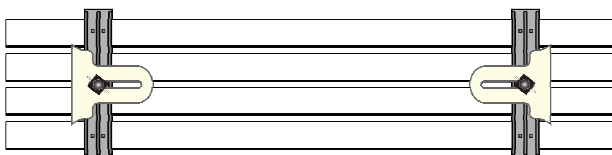
fig. 1

2. Loosen off the wing nuts holding the wing nuts holding the cleats in position (see fig. 2)
3. Turn the board over so that the slats are facing upwards.
4. Place the board over the bath traversing its width.
5. Reach underneath the board and push both cleats outwards evenly until they are gripping the wall of the bath.
6. Tighten off the wing nuts. N.B. It is only necessary to tighten the wing nuts finger tight. Do not over tighten them as this can cause the threaded brass insert to come loose, which could cause the product to fail.
7. Double check the cleats are friction tight to the wall of the bath before use.

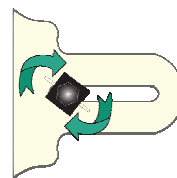
fig. 2

Always check the tightness of the board before each use

Always repeat the installation every time the board is removed

Always make sure the correct width of board is installed to suit the bath onto which it is to be fitted.

FITTING BATH SEATS

1. Firstly, ensure that the base of the bath is free from grease or any form of obstruction .and is perfectly clean.
2. Take the bath seat and dampen the four suction pads on the underside of the seat (see fig. 3).

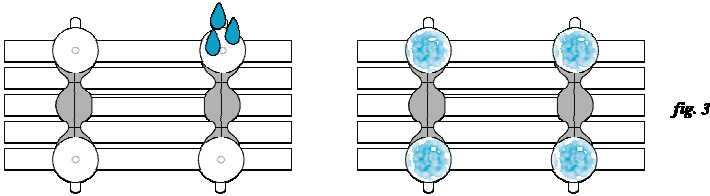
fig. 3

3. Place the bath seat onto the base of the bath (normally at the opposite end to the taps) with the slats of the seat traversing the width of the bath.
4. Push down firmly on the flaps ensuring that all four flaps make full contact with the base of the bath (see fig. 4)

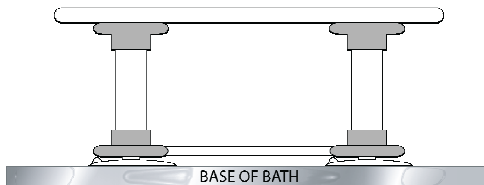
fig. 4

5. Check that the bath seat is secure by trying to lift the seat without disengaging the flaps

Always check the tightness of the seat before each use
Always repeat the installation every time the seat is removed

6. Should you wish to remove the bath seat for any reason, this can be done by simply lifting the ears of the 4 suction caps (see fig. 5), this will cause the suction to be released. To refit, repeat the fitting instructions above.

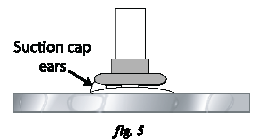
fig. 5

The bath seat can move sideways until downwards pressure is applied to the seat. It may, therefore, be advisable to transfer to the bath seat from a bath board.

CLEANING

Only use a mild detergent or disinfectant, never use an abrasive cleaner. You may autoclave all Medina products up to a temperature of 85°C. Always ensure that you wipe dry all equipment and treat joints with a water dispersing agent, e.g. WD40 after washing.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety. This includes tightness of all nuts/bolts/castors and other screw in/bolt in/push in components.

Replace all worn, missing or damaged parts where necessary. If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

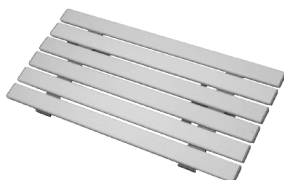
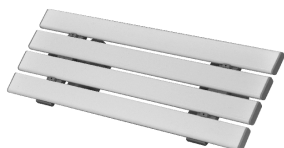
Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai Germany (Altai Europe Ltd.)